

TuffTask® Usage Guide

High performance creped wipe used in a variety of cleaning applications



1. Access wipes using perforated panel

Gently pinch one side of the perforated top and peel back the plastic to expose the dispensing area.



2. Pull the first wipe from the dispensing area.

Wipes are designed for single-wipe dispensing to minimize cross-contamination.



3. Use the wipe as needed for cleaning, drying or surface preparation.

The wipe can be used with a compatible cleaning solution. Always follow the chemical manufacturer's instructions.



4. Wipe surfaces with unidirectional strokes, going in the same direction each time.

Slightly overlap each stroke to ensure even spill or debris pickup and avoid creating any missed areas.



5. Use all clean sides of the wipe.

Get a new wipe when there is no clean, usable area left. Always change the wipe when it becomes excessively soiled.

NOTE: Use different wipes for patient bathrooms.



6. Dispose of the used wipe in the trash.

Always follow the proper biohazard waste disposal procedures (blood, bodily fluid, etc.)

DISCLAIMER: The following document is intended to serve as a tool to provide documentation of competency only and is in no way providing certification of the information in this document. Facilities should follow their own guidelines, policies and procedures which, in all cases, will take precedence over this training document.