

# SorbMore® RapidRescue™ Absorbent Pad Usage Guide

Engineered for Rapid Response and Safer Environments




## 1. Identify the Spill Area Quickly

Approach spills - whether water, oil-based, or bodily fluids - immediately upon discovery. Minimize foot traffic around the area and communicate the hazard to nearby staff.




## 2. Don Proper Personal Protective Equipment

Before engaging with the spill, wear gloves, gown, and other PPE as per your facility's infection control protocol to ensure personal and environmental safety.

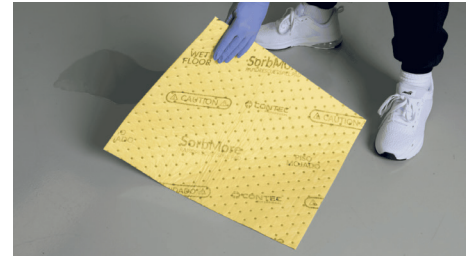

## 3. Deploy SorbMore® RapidRescue™ Pad over the Spill

Place the SorbMore RapidRescue pad (SORB0016) print side up on the spill area. The cautionary print will provide an extra warning for passers-by. Be sure to follow any additional spill mitigation safety protocols outlined by your facility.

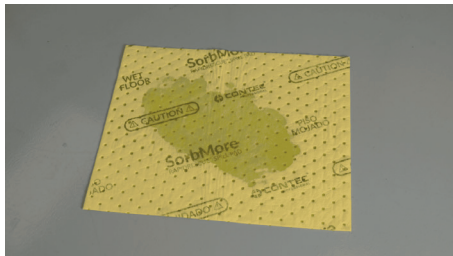

## 4. Let the Pad Absorb the Liquid Fully

The high-absorbency design allows the pad to soak up water-and oil-based liquids quickly. The bright yellow surface and bilingual "Caution Wet Floor" warning alerts staff and patients to potential risk.



## 5. Safe Removal and Disposal

Once the pad is saturated, lift it from the corners without touching the absorbed liquid. Follow your healthcare facility's protocols for biohazard or general waste disposal.



## 6. Final Surface Disinfection

Dispose of the pad in an approved garbage receptacle. If applicable, use a hospital-grade disinfectant to clean the surface area where the spill occurred.

*DISCLAIMER: The following document is intended to serve as a tool to provide documentation of competency only and is in no way providing certification of the information in this document. Facilities should follow their own guidelines, policies and procedures which, in all cases, will take precedence over this training document.*